COD. 227780

Rev. 01

Data 2025/11

LINEA MENG

Prodotto SAO

Tipo Leaflet

FUSTELLA 225x200 mm

COLOR K

DE

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EN

PROPERTIES AND INDICATIONS

Saline solution is suitable for rinsing all types of contact lenses. pH balanced formula similar to that of human tears, mild on the eye.

INSTRUCTIONS FOR USE

Wash, rinse and dry your hands before handling the lenses. Saline can be used to rinse contact lenses after cleaning, just before disinfection with disinfecting solutions. After disinfection, before application, the lenses can also be rinsed with saline as an alternative to multipurpose solutions. Close the bottle after use.

PRECAUTIONS

Discard contents of bottle 90 days after opening. Do not use if tamper-evident seal is damaged. Store between 5 and 30°C. Do not use the product after the expiry date.

WARNINGS

In the event of persistent eye irritation, discontinue use immediately, remove the lens from the eye and consult your contact lens practitioner. Keep all contact lens care products away from children. Do not allow the nozzle tip to come into contact with any surfaces. Do not rinse the contact lenses or lens case directly with tap water. Never swallow the saline solution. Any serious incidents that occur in relation to the device should be reported to Soleko S.p.A. and the competent authority of the Member State in which the user is established (vigilance@soleko.it).

CONTRAINDICATIONS

Do not use the product if you are allergic or sensitive to any of the ingredients.

CONTENTS

Sodium Chlorite 0.005%, Sodium Chloride, Dibasic Sodium Phosphate, Monobasic Sodium Phosphate, Disodium EDTA, Purified water.

KEY: SYMBOLS USED ON THE PRIMARY PACKAGING, SECONDARY PACKAGING, INSTRUCTIONS FOR USE			
	Manufacturer		Symbol of Medical Device
	Lot Number		Consult instructions for use
	Sterilised using aseptic processing techniques		CE marking
	Use-by date		Recycle
	Date of manufacture		Storage/Transport conditions

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