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EN**PROPERTIES AND INDICATIONS**

Saline solution for rinsing all types of soft and rigid gas-permeable contact lenses. The solution is pH-balanced and very compatible in the eye. It safely removes loosened debris and traces of daily cleaner. Please read instructions carefully before use.

USES

Saline solution can be used as a rinse after cleaning or disinfecting, to rinse lens cases and before re-insertion on the eye.

DIRECTION FOR USE

Before handling the contact lenses, wash your hands thoroughly. Saline solution can be used to rinse contact lenses before disinfecting lenses with cleaning solutions. After disinfecting, saline solution may be used before placing lenses into the eye as an alternative to rinsing with multipurpose solutions.

PRECAUTION

Once open, use within 60 days. Make sure the dropper tip is sealed before first use. If already open, do not use. Store between 5°C and 30°C! Do not use after the expiry date. Never rinse contact lenses with tap water. Never use saline solution for lens disinfection. Do not use the product if you are allergic or sensitive to any of the ingredients. Do not use eye medication whilst wearing contact lenses, except under medical supervision. Keep out of reach of children. **DO NOT SWALLOW!**

CONTRAINDICATIONS

No contraindication reported. If you experience any irritation or any other adverse effects, remove lenses and consult your specialist.

INGREDIENTS

Ethylene diamine tetra acetic acid sodium salt 0.01% (Stabilizing), Sodium Chlorite 0.005%, Sodium Chloride 0.79g/100ml, Dibasic sodium phosphate, Monobasic sodium phosphate, Purified water.

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